

SONIC | Velocity™



Introducing sonicBLU

SONIC
innovations



sonicBLU: Hands-Free Headset for Mobile Phones

- Uses Bluetooth® to wirelessly send and receive calls from Bluetooth-enabled phones
- When a call is received, sonicBLU streams the audio from the phone directly to the hearing instruments – no feedback, no interference
- When fit binaurally, sonicBLU streams audio to *both* hearing instruments simultaneously





Specifications

Specifications	
Talk Time	Up to 4 hours
Stand-by-Time	Up to 70 hours
Operating Range	Up to 10 meters
Power Consumption	27mA (working) <1mA (stand-by)
Battery	Rechargeable Lithium-ion 120mAh
Charging Time	1.5 to 2 hours

sonicBLU Main Unit

1. Charger Connector
2. Volume Control (Increase)
3. Volume Control (Decrease)
4. Indicator Light
5. Multi-Function Button
6. Microphone



24

12

6

4

Charging sonicBLU

- Connect the charger to the wall outlet
- Connect the charger cable to sonicBLU
The red indicator light is displayed during charging. If charging does not start, disconnect the charger and then plug it in again. Fully charging the battery may take up to 2 hours.
- Completion of Charge
When the battery is fully charged, the red indicator light turns off.

24

12

6

4

Turning sonicBLU On or Off

To switch on:

- Press and hold the multifunction button for 3 seconds. The blue indicator light will flash.

To switch off:

- Press and hold the multifunction button for 3 seconds. The blue indicator flashes rapidly and then shuts off.





Pairing sonicBLU with a Bluetooth Phone

- With sonicBLU and the phone in proximity:
 1. With the sonicBLU off, hold the multifunction button until the indicator light continuously flashes blue-red (about 5 seconds).
 2. Perform a device discovery from the phone (for details, refer to the user guide that came with the phone). When the discovery is complete, the phone displays a list of found Bluetooth-enabled devices.
 3. Select “sonicBLU” from the list and follow any prompts.
 4. When prompted, enter “0000” as the password and confirm. When the pairing is complete, the blue indicator flashes rapidly.
- The phone is now paired with sonicBLU!

Using sonicBLU with the Neckloop

1. Charge the sonicBLU battery before use
2. Place the neckloop around the neck
3. Plug the connector on the neckloop into the sonicBLU main unit
4. Pair the sonicBLU with a Bluetooth phone
5. To accept a call
 1. Select the hearing aid's telephone program
 2. Press the multifunction button



Using sonicBLU with the Earhooks

1. Charge the sonicBLU battery before use
2. Place the hooks on the ears next to the hearing aids
3. Plug the connector on the earhook cable into the sonicBLU main unit
4. Pair the sonicBLU with a Bluetooth phone
5. To accept a call
 1. Select the hearing aid's telephone program
 2. Press the multifunction button

